


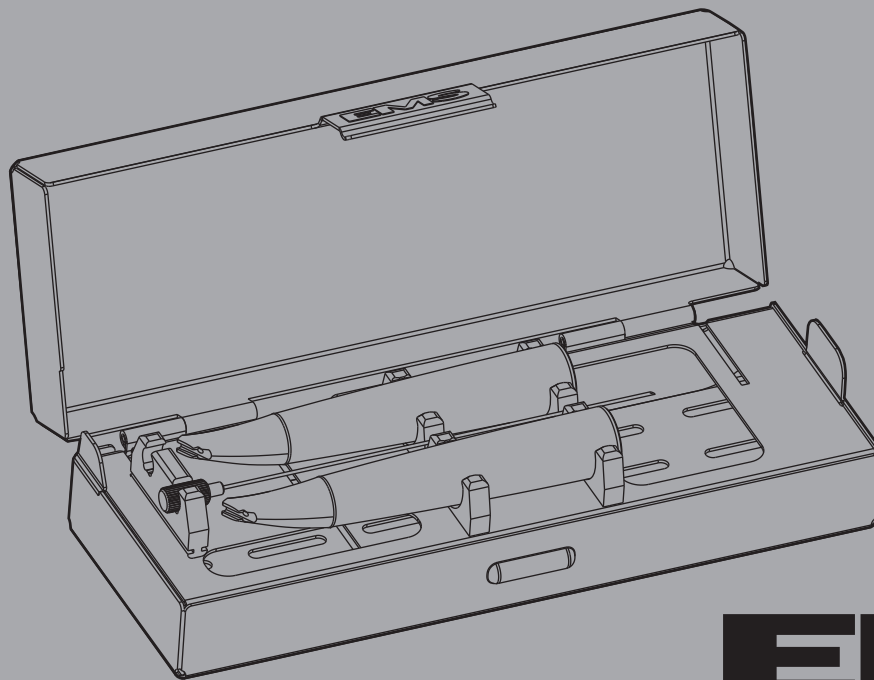
OPERATION INSTRUCTIONS
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
INSTRUCCIONES DE EMPLEO
ISTRUZIONI PER L'USO
BRUKSANVISNING

PERIO-FLOW® System

Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

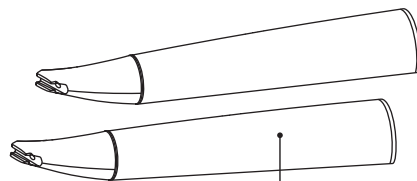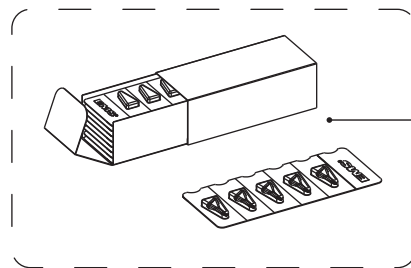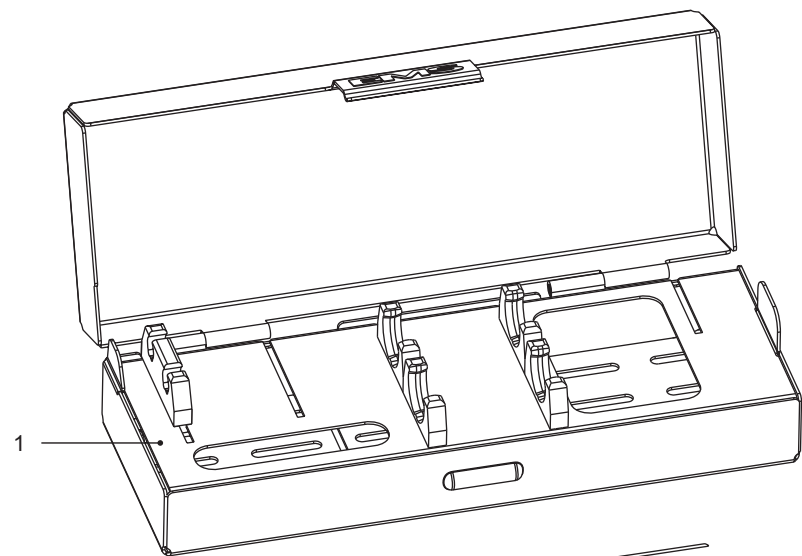EMS 
ELECTRO MEDICAL SYSTEMS

1

2

3

4

5

COMPONENTS

- 1 SystemBox PERIO-FLOW®
- 2 EasyClean
- 3 Cleaning needle
- 4 Handpiece
- 5 Nozzles

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operation instructions or contents of the set due to technical or scientific improvements.

DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

This system is for use in dentistry with EMS AIR-FLOW units.

This product must only be used by a qualified and trained personnel (dentist or dental hygienist).

This product should only be used with powders supplied by EMS for a subgingival application.

This product remove the subgingival biofilm during periodic periodontal treatments. Its long fine nozzle was specifically developed for easy access to the periodontal pocket.

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PLEASE READ THIS BEFORE BEGINNING WORK!

This operation instructions are to ensure the correct installation and use of this product.

Please read these operation instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger

Risk of injury

Caution

Risk of property damage or environmental harm

Please note

Useful additional information and hints

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Tillåtet

About this manual

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

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EMS

CONTENTS

General hints for treatment.....	2
General information.....	2
Preparing for treatment.....	6
Drying the sterilized handpiece.....	6
Nozzle installation and handpiece connection.....	8
Usage Precautions.....	10
Wear a mask and eye protection.....	10
Treatment.....	14
Power setting air/powder and water flow rate adjustment.....	14
Positioning of the saliva pump and the high speed evacuator suction nozzles.....	16
Working Technique.....	18
Precautions to be taken at the end of the treatment.....	24
Removing the nozzle and the handpiece.....	26
Cleaning, disinfecting and sterilizing.....	28
Cleaning the handpiece cord.....	28
Cleaning the handpiece with needles.....	30
Cleaning the handpiece with the Easy Clean.....	32
Sterilization.....	37
Drying and connecting the sterilized handpiece.....	38
Safety precautions.....	40
Storing the product.....	44
Product disposal.....	46
Warranty.....	46
Accessories.....	46
EMS-Service.....	48
Symbols.....	50
Technical data.....	51
Troubleshooting.....	52

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General information

Indication:

All patients suffering from periodontal disease and peri-implantitis.

This system is specially recommended for:

- Periodontal maintenance of deep pocket up to 10 mm following initial treatment
- Periodontal biofilm removal
- Implant cleaning

Contra-indications:

 The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients:

- Endocarditis
- Pregnancy, breast feeding
- Contagious disease
- Immune deficiency (neutropenia, angranulocytosis, diabetes, hemophilia)
- Patients under treatment (radiotherapy, chemotherapy, antibiotics)

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 The air jet and powder may cause breathing difficulties. Please apply appropriate restrictions for the treatment of risk patients: Patients suffering from chronic bronchitis or asthma must not be treated under any circumstances with this product.

 Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

 The nozzle cannot be sterilized.

 The single use nozzle must be used for one single patient only. Never reuse a nozzle because treatment will be ineffective and the risk of emphysema would increase.

 The use of any other powder than the EMS powders for subgingival application would reduce the nozzle's service life. As a result the treatment would become ineffective and would increase the risk of emphysema.

For more details about how to use the AIR-FLOW units and the EMS powder, please read their operation instructions.

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 Following the sterilization, blow the interior of the handpiece with compressed air to eliminate possible humidity.

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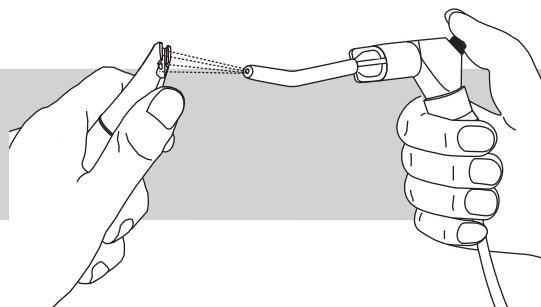
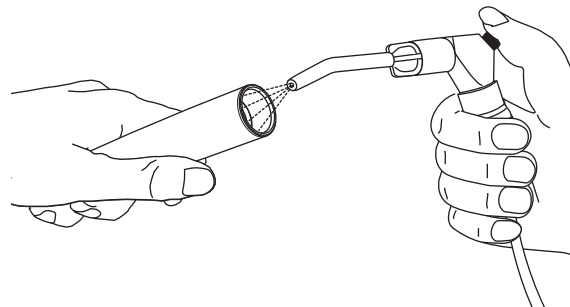
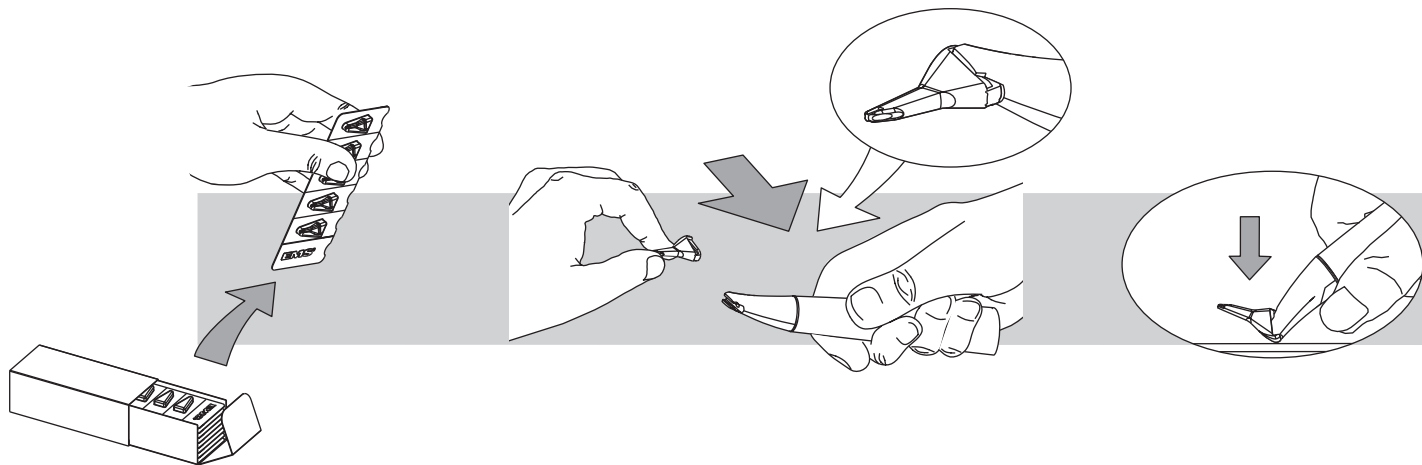
Nozzle installation and handpiece connection

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 Before each use, make sure the nozzle is correctly attached: The nozzle must be fully inserted.

 Do not use the nozzle in case of doubt that package seal was broken, or that package or product is damaged.

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 A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.

 To limit the risk of bacterial or viral infections and inhalation of powder, we recommend to the dentist and the hygienist to wear a safety protection mask.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

 The PERIO-FLOW nozzle should never be pointed toward the patient before, during and after operation.

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TREATMENT

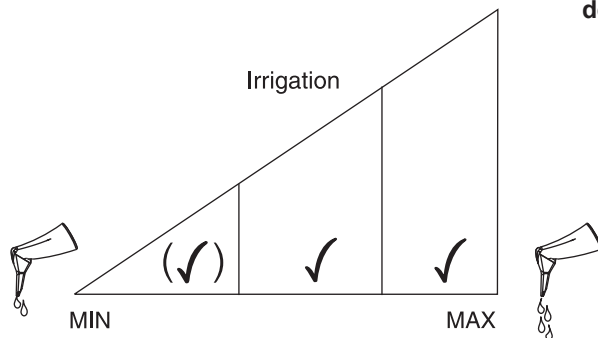
Power setting air/powder and water flow rate adjustment

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Never use the PERIO-FLOW handpiece without water flow; this could cause lesions in the treatment area.

- Start the treatment with water flow rate adjustment control set at medium setting. Then adjust the irrigation to obtain a flow as required for treatment.
- Start the treatment with the power setting air/powder control to the minimum setting. It can then be adjusted as required during the treatment.

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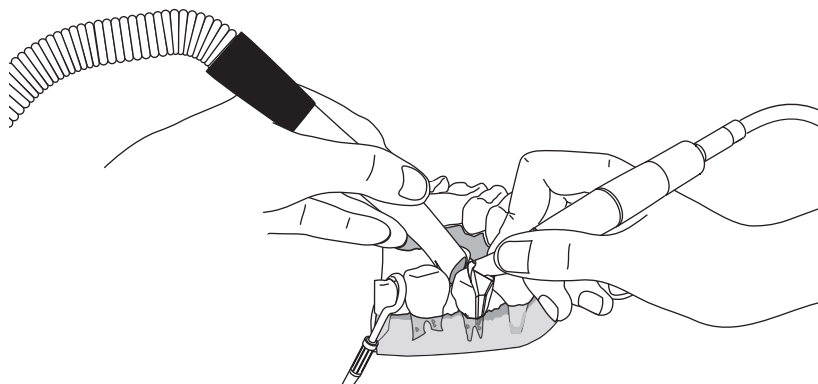
Working Technique

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 The nozzle length enables cleaning pockets over 5 mm deep.

 Never treat with PERIO-FLOW the pockets down to the apex. At least 3 mm of bone must be present to prevent the risk of emphysema.

 Always use an evacuator during treatment.

1. Insert the nozzle into the periodontal pocket. The penetration depth depends on the pocket depth. Do not force the nozzle into the pocket because this could hurt soft tissues.

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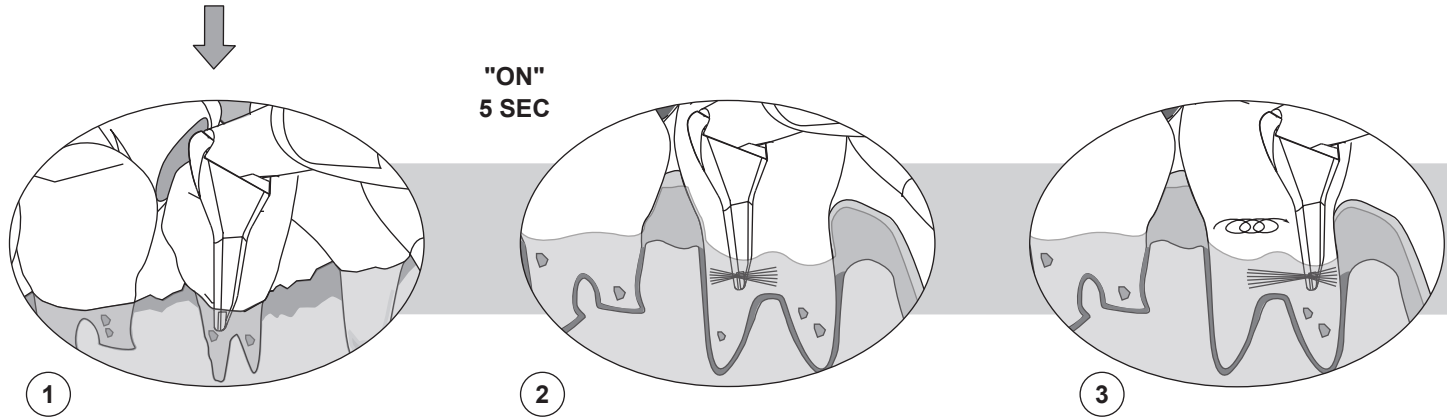
2. When the nozzle is inserted in the pocket, press the pedal of your device for 5 seconds. EMS powder/air mixture confined by a water jet is sprayed into the periodontal pocket. Due to the mechanical action of the air, powder and water spray, the subgingival biofilm is eliminated from the root surface.

3. Place the nozzle on the edge of the sulcus and let the spray act for 5 seconds maximum on each side of the tooth (vestibular, lingual, distal).

Carry on the treatment with short tiny rotary motions of the nozzle.

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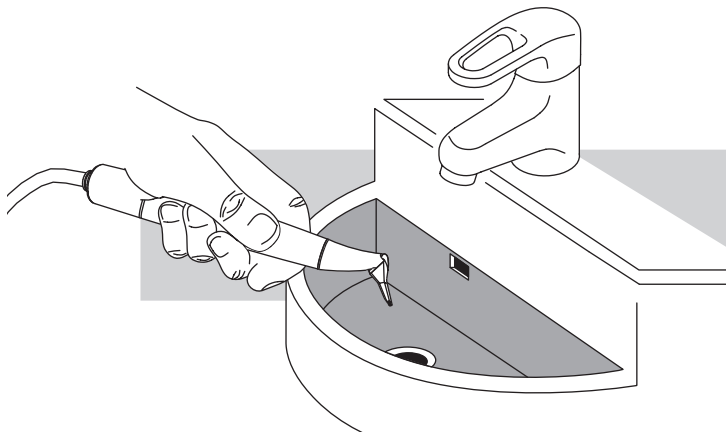
Precautions to be taken at the end of the treatment

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Immediately after treatment, point the nozzle to a sink and turn off the device. During shut off an air spray will remove eventual residues of powder from the air channels.

 The automatic cleaning of the air system is also activated in case of power failure. This is why the PERIO-FLOW nozzle should never be pointed toward the patient before, during and after operation.

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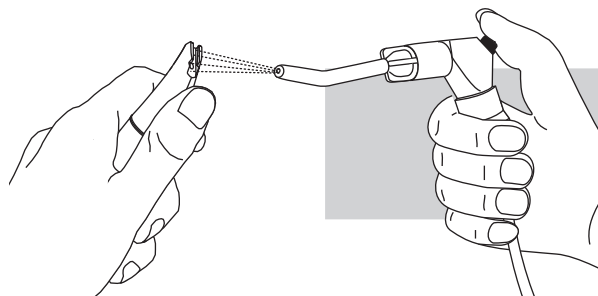
Cleaning the handpiece with needles

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
AIR

AIR

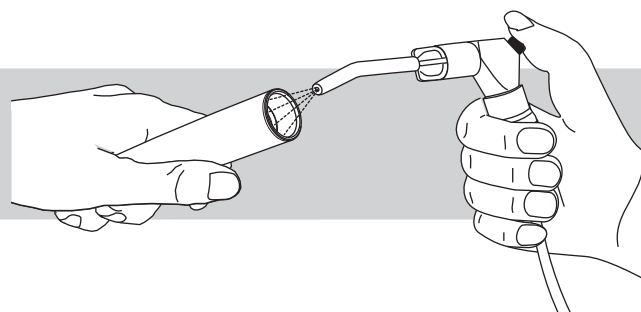Remove the residual powder present in the tubes by using the cleaning needles. Proceed with precaution and do not use force because the needles can easily break. Use only the tools provided.

 Please follow the recommendations of the Reprocessing Instructions manual delivered with your product regarding the procedures for cleaning, disinfecting, sterilizing and packing of the components.

 Clean the handpiece with running water before disinfection.

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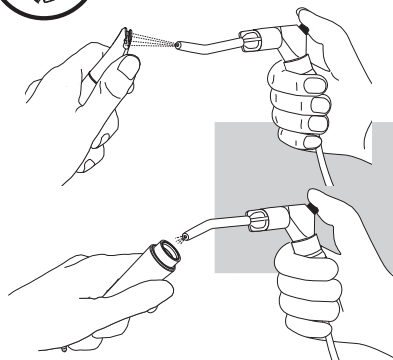
Cleaning the handpiece with the Easy Clean

The EASY CLEAN is designed to unclog and to facilitate the cleaning of the EMS AIR-FLOW handpieces.

 The EASY CLEAN can be thermal disinfected and also sterilized at up to 135°C in the autoclave.

Periodical cleaning of the handpiece using the EASY CLEAN.

Rinse through central lumina in the normal direction of flow (no back rinsing) using a disposable syringe filled with more than 2 ml of drinking water quality applied to the purge adaptor.

After removing the EASY CLEAN, blow compressed air into the handpiece until completely dry inside the handpiece.

 It is recommended to perform a periodical cleaning once a week or after 15 treatments.

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Unclogging the handpiece using the EASY CLEAN

To get more force on the pressure use a small syringe (1 or 2 ml) filled with drink water.

Insert the assembled syringe with the EASY CLEAN into the handpiece lumina.

Press strongly if needed and maintain the pressure until the handpiece is unclogged (it can take 2 to 5 seconds) The water passing through the handpiece will dissolve completely the remaining powder inside the lumina and ensure an optimal function.

After unclogging the handpiece, apply the periodical cleaning.

 Unclog the handpiece before sterilizing.

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Sterilization

Please follow the recommendations of the Reprocessing Instructions manual delivered with your product regarding the procedures for cleaning, disinfecting, sterilizing and packing of the components.

Put the accessories in sterilization bags or use the EMS SystemBox. The SystemBox allows the conditioning of the accessories for sterilizing. Different configurations are available, please ask your dental dealer. To maintain the sterility when stocking the equipment, you can put the SystemBox in a sterilization bag.

After sterilizing without a sterilization bag, stains may appear on the SystemBox.

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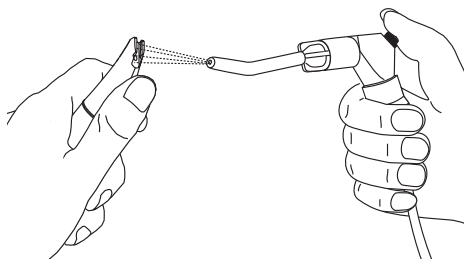
**Drying and connecting the sterilized
handpiece**

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
AIR

AIR

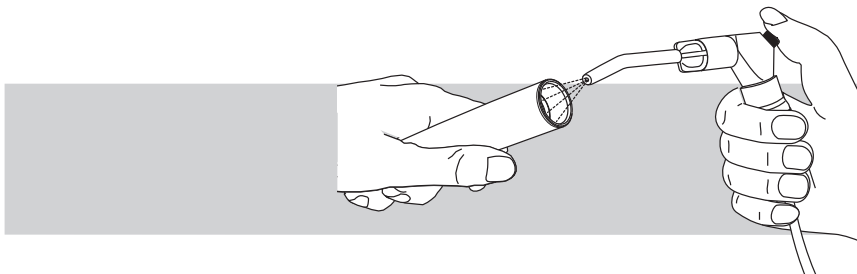
⚠ After sterilization, moisture may be present in the handpiece. Blow the interior of the handpiece with dry compressed air. Make sure the connection is dry.

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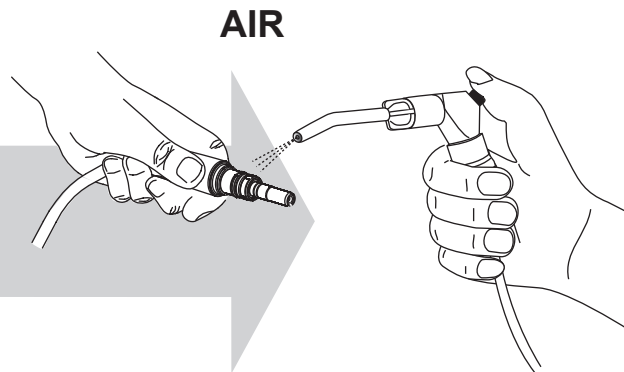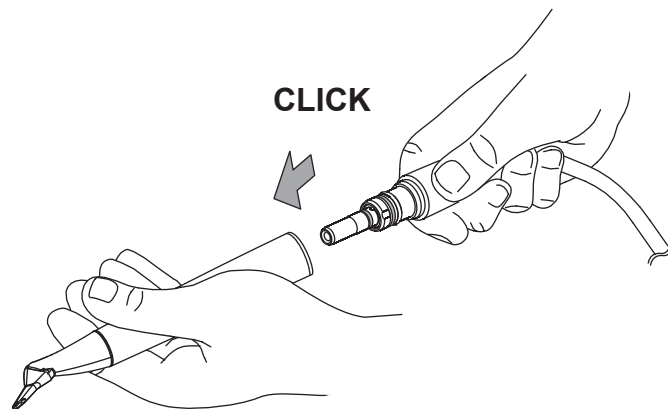
SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operation instructions, or improper preparation and maintenance.

 Use for the intended purpose only: Before using the product, make sure that you have studied the operation instructions. This also applies to any equipment used with this product. Failure to observe the operation instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Before each use, clean, disinfect and sterilize the different pieces and accessories of this product (nozzles excepted). Please refer to the information provided in the operation instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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Use only the powder recommended by EMS for the subgingival application when the PERIO-FLOW handpiece is connected to the unit. Any other powder could cause premature wear of the nozzle and would consequently increase the risk of emphysema and damage to the unenameled dentine.

Always examine the product for damage before starting the treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

Have this product exclusively repaired by an approved EMS repair center.

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STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing".
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

 The powder should not be stored at the proximity of a heat source.

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PRODUCT DISPOSAL

The product, its accessories and its packaging do not contain any substances that are dangerous for the environment.

Should you wish to definitively dispose the product, please comply with the regulations which apply in your country.

WARRANTY

The warranty is valid from the date of purchase of your product for the period indicated in the Registration Card.

Damages due to non-adherence to the operation instructions or wear out of parts are excluded from warranty.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operation instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operation instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

DESCRIPTION

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	PERIO-FLOW®
Classification 93/42 EC	Class IIa
Operation conditions	+10°C to +40°C 30% to 75% relative humidity
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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TROUBLESHOOTING

Symptom	Solution
No water spray is coming from the nozzle	• Remove the nozzle and verify that a water jet comes out from the handpiece • Check the water adjustment on your unit • Check the connections between the unit, the cord and the handpiece • Check the water supply • Check for presence of water at the cord outlet port • Check for presence of water at the system connector outlet without cord • Send the system to an approved EMS repair center
No powder/air spray is coming from the nozzle	• Remove the nozzle and verify the powder/air come out from handpiece • Remove any residue with the cleaning needle or the Easy Clean in both water & air channels • Check the compressed air supply • Check for presence of powder/air at cord outlet port without handpiece • Check for presence of powder/air at the system connector outlet without cord • Check the quantity of powder in your unit • Change the nozzle • Send the system to an approved EMS repair center
Water leakage between the handpiece and the nozzle	• Check the position of the nozzle and its position on the handpiece • Check for residues between the nozzle and the handpiece • Check that nothing affect the alignment of the nozzle • Change the nozzle • Change the handpiece • Send the system and the handpiece cord to an certified EMS repair center
Water leakage between handpiece cord and handpiece	• Check the condition of the handpiece connecting o-rings • Send the system and the handpiece cord to an approved EMS repair center
The efficiency diminishes	• Check the quantity of powder in your unit • Eliminate residual powder present in the channels using cleaning needle or the Easy Clean • Change the nozzle • Send the system and the handpiece cord to an approved EMS repair center
If the handpiece falls	• Replace the nozzle and check the holding of the nozzle • If the nozzle doesn't hold or it is not waterproof, send the handpiece to an approved EMS repair center

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e-mail: info@ems-ch.de

SPAIN

EMS Electro Medical Systems España SL

Bernardino Obregón 14 bis

E-28012 Madrid

Tlf. +34 91 528 99 89

Fax +34 91 539 34 89

e-mail: administracion@ems-espana.com

FRANCE

EMS France Sarl

23, Av. Louis Bréguet

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F-78140 Vélizy Villacoublay

Tél. +33 1 34 58 03 80

Fax +33 1 34 58 03 90

e-mail: info@ems-france.fr

ITALY

EMS Italia S.r.l

Via Faravelli 5

I-20149 Milano

Tel. +39 02 3453 8111

Fax +39 02 3453 2778

e-mail: dental@ems-italia.it

USA

EMS Corporation

11886 Greenville Avenue #120

Dallas, TX 75243, USA

Tel. +1 972 690 83 82

Fax +1 972 690 89 81

e-mail: info@ems-na.com

EMS - SWISSQUALITY.COM