
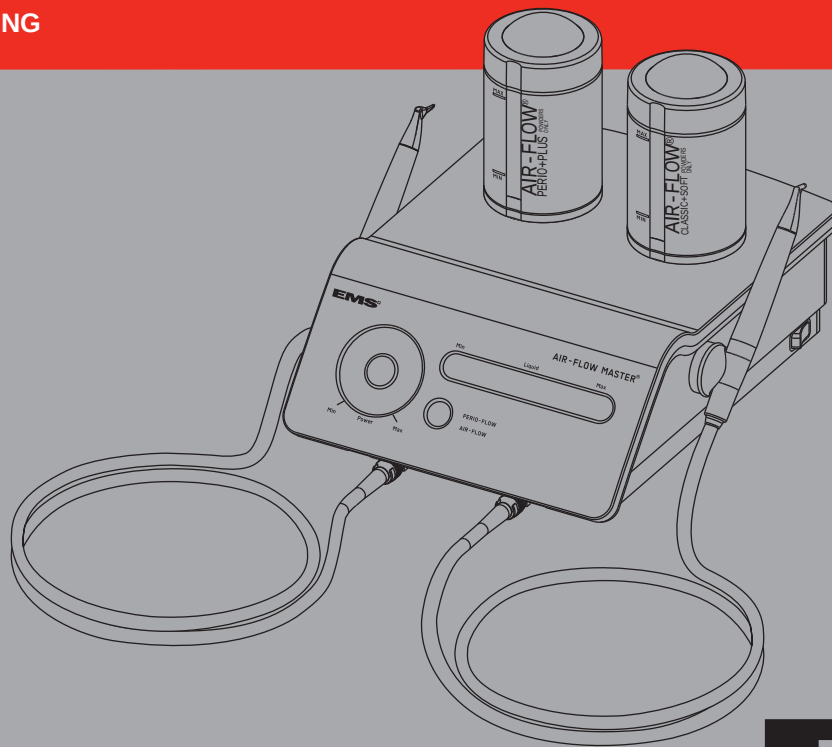
OPERATION INSTRUCTIONS
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
INSTRUCCIONES DE EMPLEO
ISTRUZIONI PER L'USO
BRUKSANVISNING

AIR-FLOW MASTER®

Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

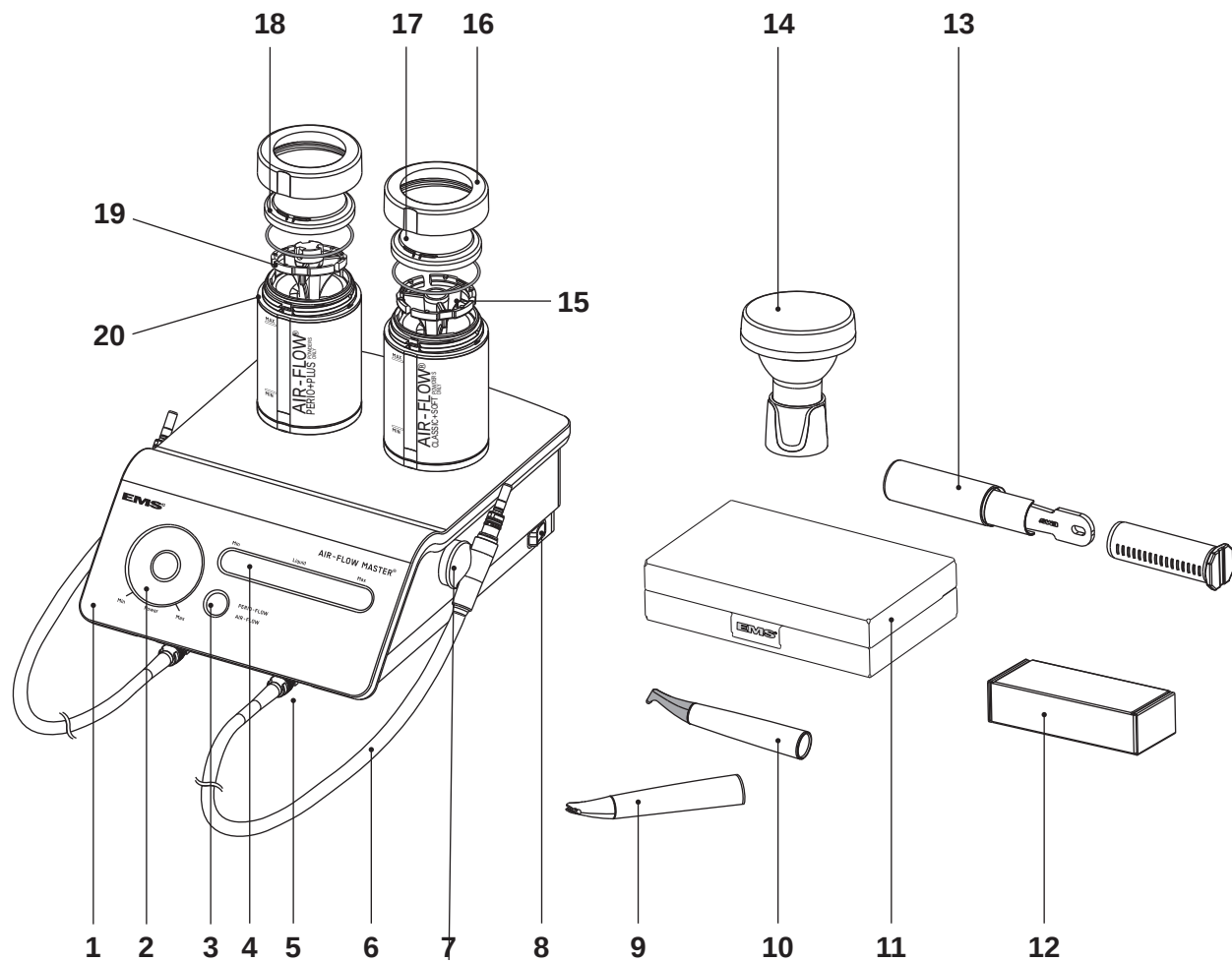EMS 
ELECTRO MEDICAL SYSTEMS

COMPONENTS

- 1 Control panel
- 2 Power setting (air/powder)
- 3 Working mode selection
- 4 Water flow rate adjustment
- 5 Connector for the handpiece cord
- 6 Handpiece cord
- 7 Magnetic handpiece support
- 8 Purge switch
- 9 PERIO-FLOW® handpiece
- 10 AIR-FLOW® handpiece
- 11 SystemBox
- 12 Box of PERIO-FLOW® nozzles
- 13 PERIO-FLOW® Nozzle extractor
- 14 Easy Fill
- 15 AIR-FLOW® CLASSIC+SOFT Powder diffuser
- 16 Cap ring
- 17 Cap dome
- 18 Cap seal
- 19 AIR-FLOW® PERIO+PLUS Powder diffuser
- 20 Inter-changeable colored O-ring

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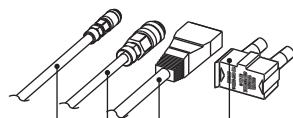
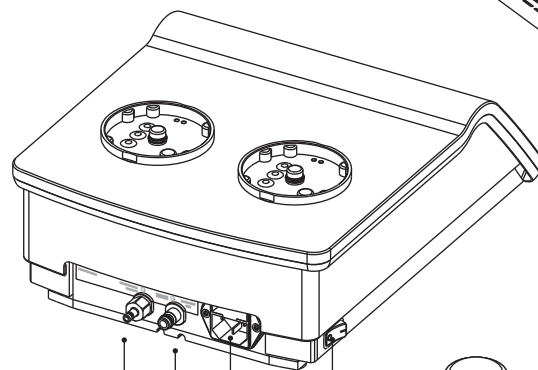
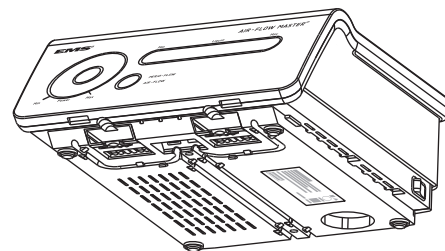
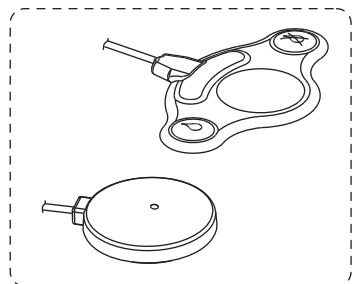
32

33

- 21 Foot pedal (2 step or multifunction depending on order)
- 22 Water supply hose
- 23 Compressed air supply hose
- 24 Power cable
- 25 Fuses
- 26 Water hose connector
- 27 Air hose connector
- 28 Power socket
- 29 Power switch
- 30 AIR-FLOW® CLASSIC powder bottle
- 31 Cap
- 32 Air filter housing
- 33 Filter cartridge

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

Your AIR-FLOW MASTER is a combination of two air polishing units.

It is for use in dentistry with EMS AIR-FLOW systems.

This product functions with the powders and the systems supplied by EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

Applications

- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding of orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Clean implant fixture prior to loading,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

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PERIO-FLOW system (optional)

This product must only be used by a qualified and trained personnel (dentist or dental hygienist).

This product should only be used with powders supplied by EMS for a subgingival application.

This product remove the subgingival biofilm during periodic periodontal treatments. Its long fine nozzle was specifically developed for easy access inside the periodontal pocket.

For further details concerning the treatments, please refer to the instruction which are supplied with your AIR-FLOW or PERIO-FLOW systems. Please do not hesitate to contact your local EMS dealer for further information.

The instructions in this manual cover the combined use of an AIR-FLOW handpiece and a PERIO-FLOW handpiece. Please refer only to instructions related to AIR-FLOW handpiece when a PERIO-FLOW handpiece is not present.

Contra-indications:

Please refer to operation instructions of the related system.

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ABOUT THIS MANUAL

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger
Risk of injury

Caution
Risk of property damage or environmental harm

Please note
Useful additional information and hints

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Vi önskar Er lycka till!

EMS

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

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EMS

CONTENTS

Installation and set up	2
Connecting the "2 step" foot pedal	2
Connecting the "multifunction" foot pedal.....	4
Connecting to a second EMS unit (optional).....	6
Connecting the unit to the water supply	8
Connecting the unit to the compressed air supply.....	10
Connecting the unit to the mains power ...	12
Electrical Safety precautions	14
Connecting the cords and placing the handpiece supports	16
Powder chambers	18
Supragingival treatment using the AIR-FLOW CLASSIC+SOFT powder chamber.....	20
Supragingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	22
Subgingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	24
Powder choice.....	26
Filling the powder chambers.....	28
Preparing for treatment.....	32
Connecting the AIR-FLOW handpiece	32
Installing the nozzle and connecting the PERIO-FLOW handpiece.....	34
Operating the unit	36
Switching On	36
Working mode selection.....	40
PERIO-FLOW mode detection	42
Setting the air/powder flow rate	44

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Setting the water flow rate	46
Before using the unit	48
Innan du använder enheten	49
Purge switch and mode.....	50
Powder exchange or refill.....	52
End of the treatment	54
Powder and water purge	54
Cleaning, disinfecting and sterilizing	56
Unit	56
Cleaning the dome	58
Cleaning the Powder Chamber	60
Maintenance.....	62
Replacing the dome	62
O- ring	64
Valve assembly	66
Replacing the O-rings of the connectors...68	
Replacing the water filter and the fuses	70
Air filter	72
Safety precautions	74
Storing the product	78
Product disposal	80
Warranty	80
Accessories.....	80
EMS-Service.....	82
Symbols	84
Technical data	87
Electromagnetic compatibility.....	90
Troubleshooting	92

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
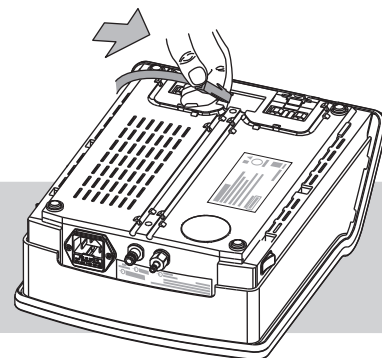
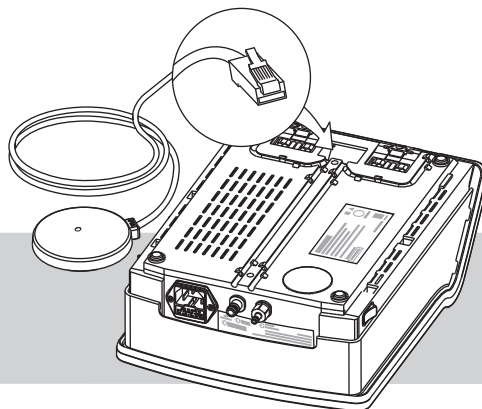
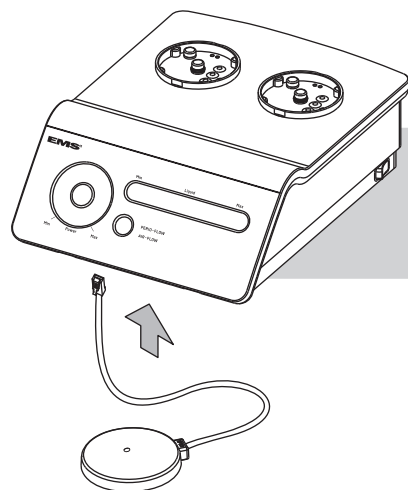
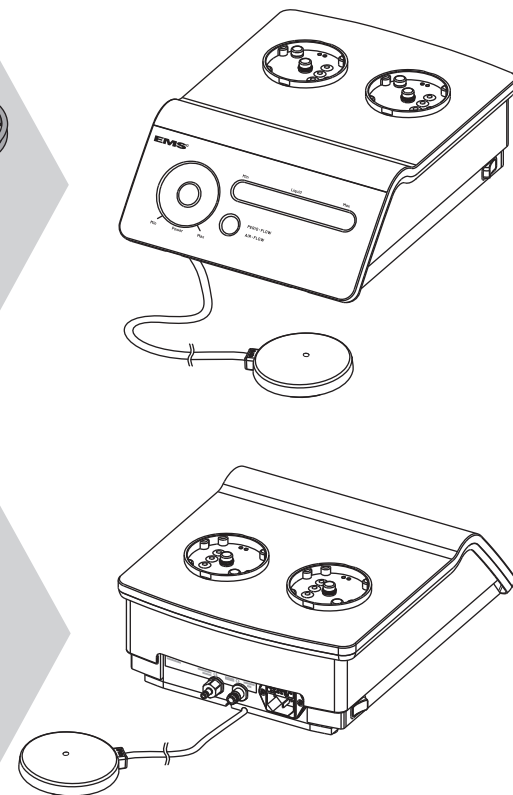
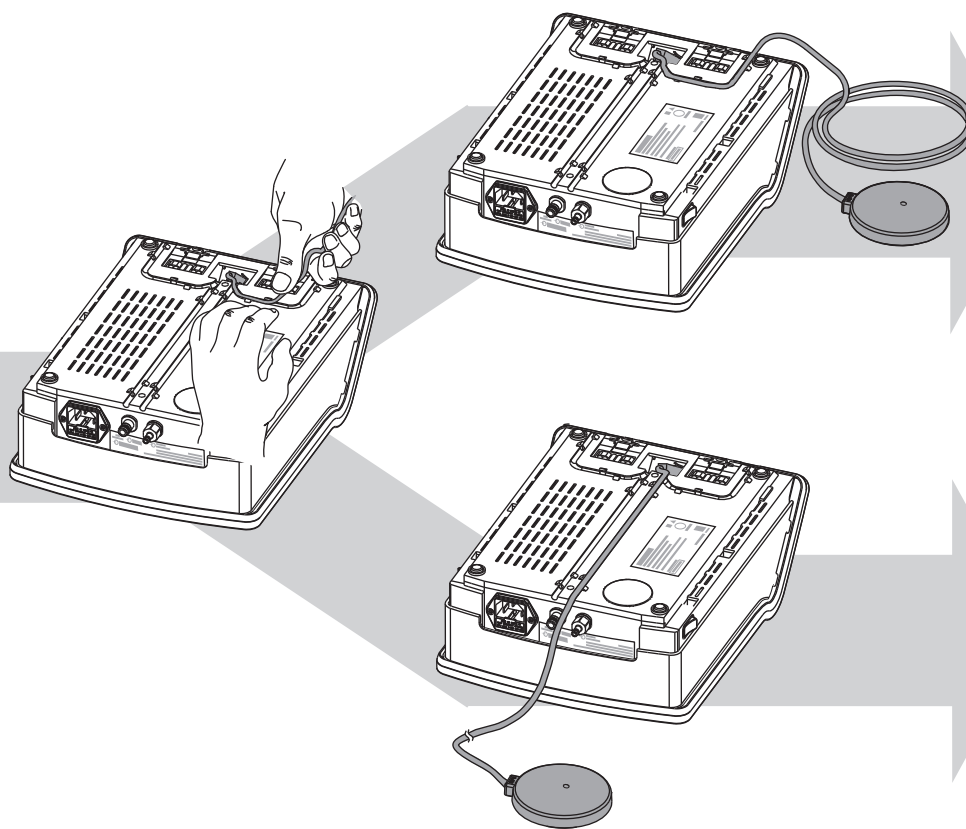
Collegamento del pedale "2 step"

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
Ansluta en "2 step" pedal

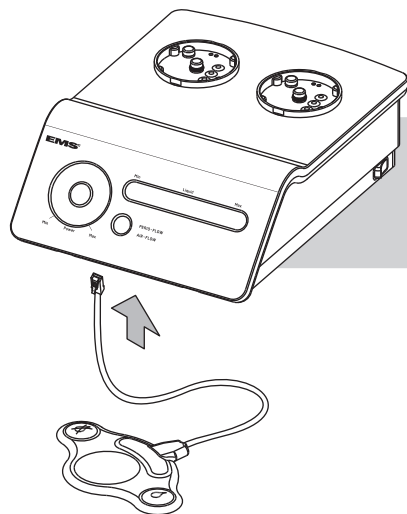
**Connecting the "multifunction"
foot pedal**

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
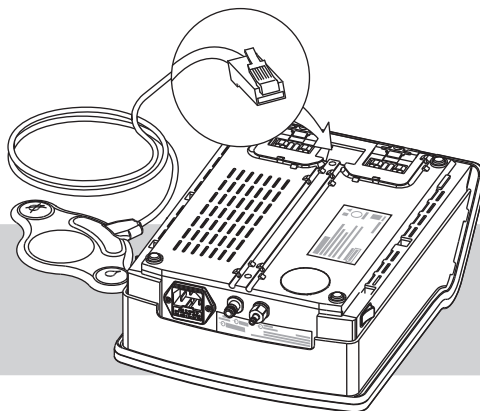
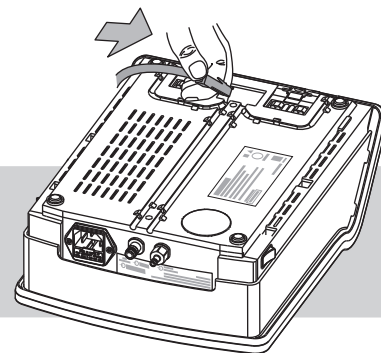
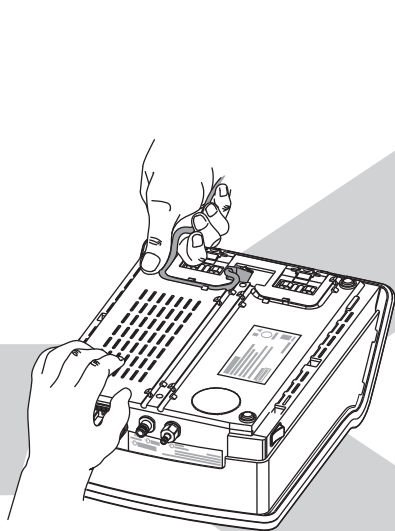
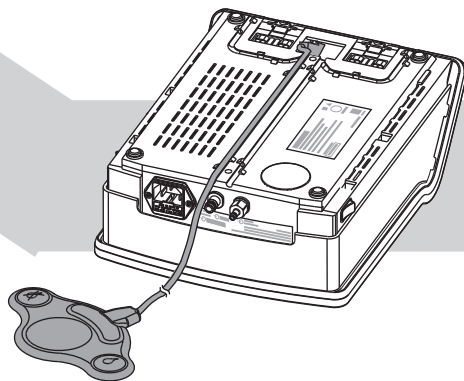
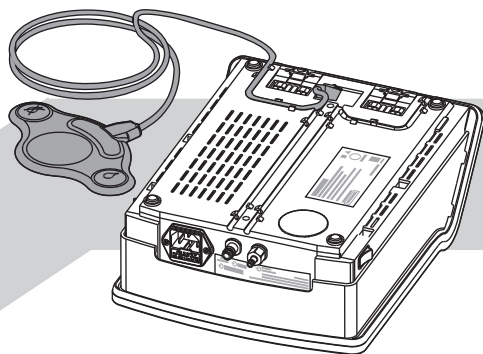
Ansluta en "multifunction" pedal

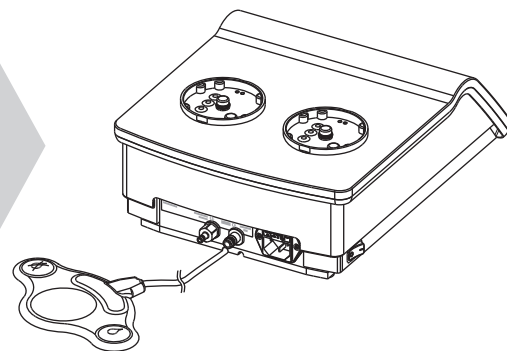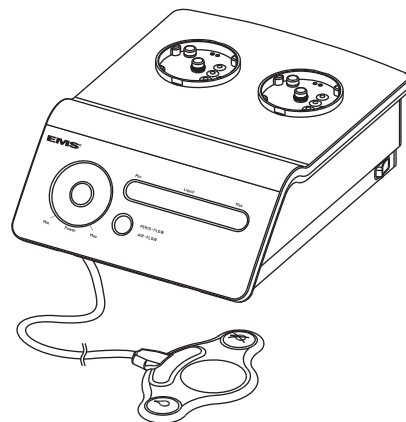Connecting to a second EMS unit (optional)

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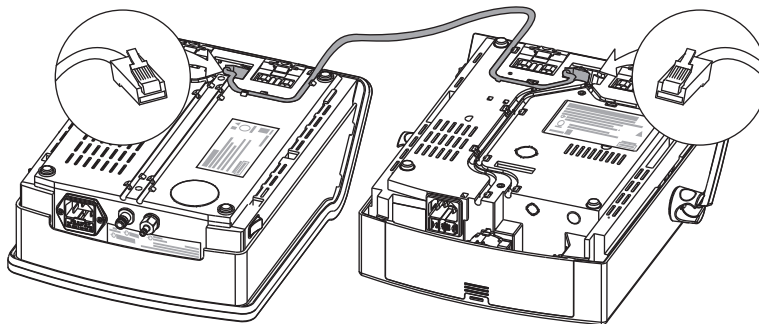The AIR-FLOW MASTER can be connected to another compatible EMS device, so both devices can be operated using a single foot pedal. A special optional link cable allows this connection. This option is only available with the "multi-function" pedal.

The pedal will trigger only the handpiece which is lifted from its support whatever the unit.

For additional details on device compatibility, please contact the distributor in your country.

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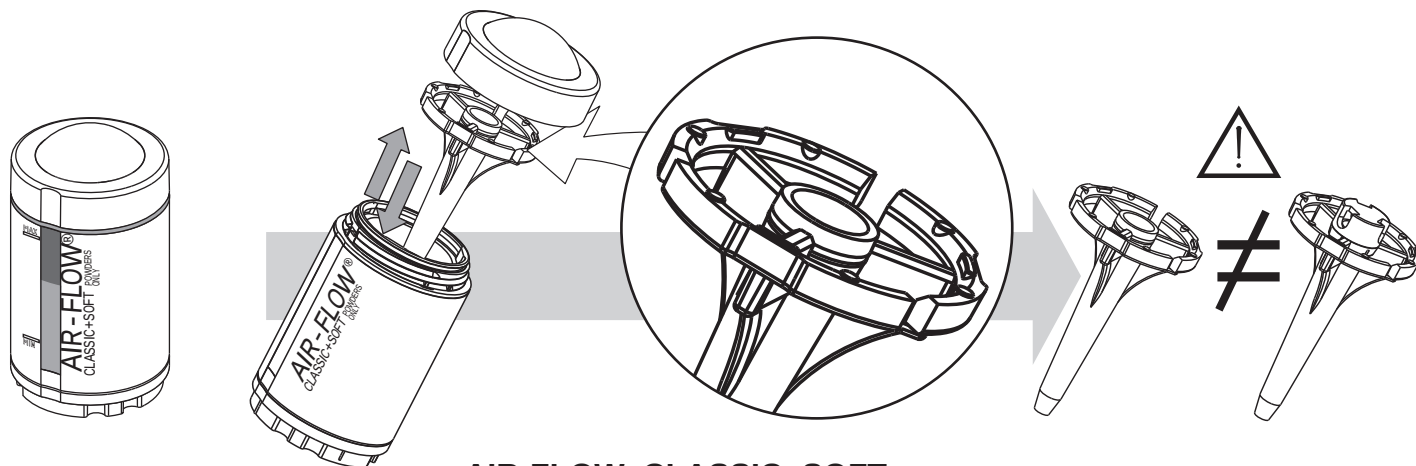
Pulverkammern

AIR-FLOW® CLASSIC+SOFT

There are two powder chambers having each a specific diffuser.

 The AIR-FLOW CLASSIC+SOFT powder chamber can be used exclusively for supragingival treatment together with the CLASSIC, CLASSIC COMFORT and SOFT powders.

 The AIR-FLOW PERIO+PLUS powder chamber can be used for both supragingival and subgingival treatments using the specific powders PLUS and PERIO.

 Do not invert the diffusers, they are not compatible.

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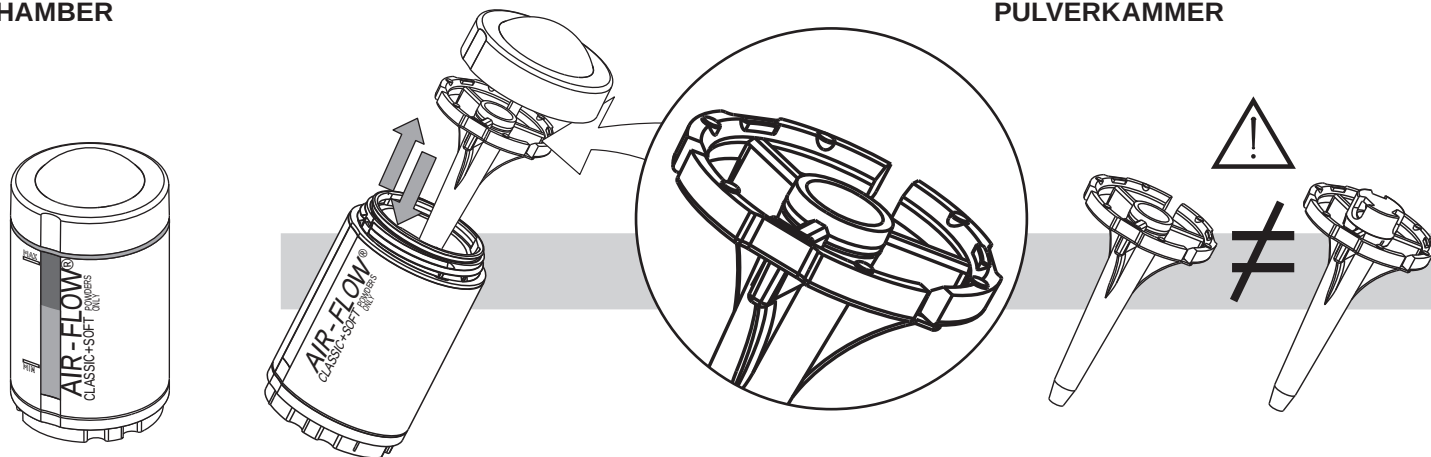
**SUPRAGINGIVAL TREATMENT
USING THE AIR-FLOW
CLASSIC+SOFT POWDER
CHAMBER**

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AIR-FLOW® CLASSIC+SOFT

The AIR-FLOW CLASSIC+SOFT powder chamber must be used only for supragingival treatments.

The AIR-FLOW CLASSIC+SOFT powder chamber can only be used with CLASSIC, CLASSIC COMFORT and SOFT supragingival powders.

Do not invert the diffusers, they are not compatible.

Do not use this powder chamber with the PERIO or with the PLUS powders.

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
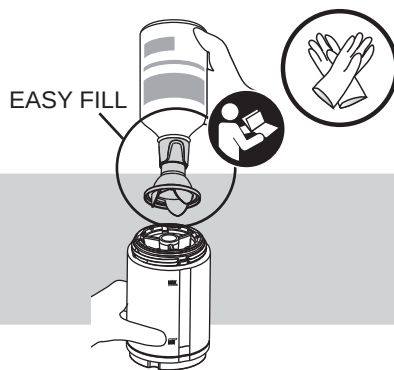
Filling the powder chambers

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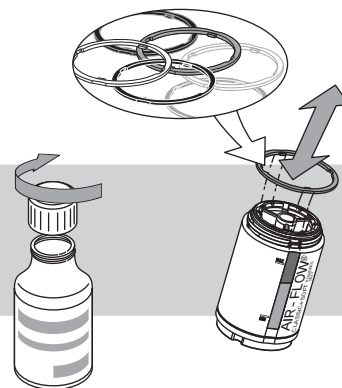⚠ Make sure that the powder chambers are absolutely dry. Moisture can cause the powder to cake and provoke a blocking of the circuit.

⚠ To avoid clogging the unit do not fill the powder chambers above the line "Max".

⚠ Use only powder provided by EMS and corresponding to the application.

⚠ Close correctly the bottle after use. The conditions of storage are described in the chapters "Storing the product" and "Technical data".

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A set of interchangeable colored rings is provided to help recognize the chosen powder flavors. Please refer to the information provided in the chapter "Maintenance".

Check the condition of the threads on the powder chamber, on the cap and on the O-ring seals. Please refer to the information provided in the chapter "Maintenance".

Clean the threads of the powder chamber and of the cap before screwing them on.

Ensure that the powder chambers are closed tight. Otherwise air and powder could leak.

Do not turn the powder chambers upside down after filling.

The PLUS powder being specially fine, tamping it down the powder chamber must be avoided when filling it.

The powder should not remain over a determined time inside the powder chamber. Read the instructions delivered with the powders.

The PLUS powder level will go down slightly a few minutes after the filling (powder compaction).

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*Temporary air pressure increase of 30% (up to 100%)
The light of the powder chamber blinks rapidly.

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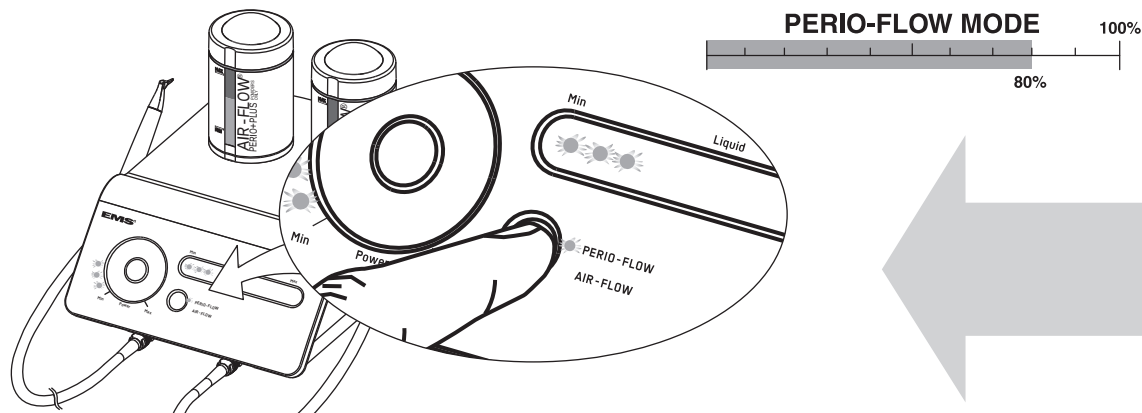
Working mode selection

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In PERIO-FLOW mode, the available air pressure is reduced compared to AIR-FLOW mode.

In PERIO-FLOW mode, one beep is generated every 5 seconds (recommended duration per single pocket treatment). Counting stops after 15 beeps, and 3 consecutive beeps will be generated. The sequence counting is initiated again when the pedal is released for more than 15 seconds.

The unit adapts the pressure when the mode is changed.

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
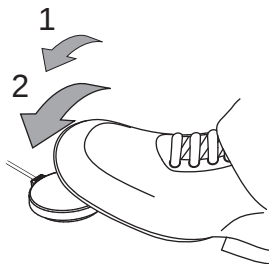
 After refilling the powder chamber it is recommended to ensure that the equipment is functioning. Make a test over a basin.

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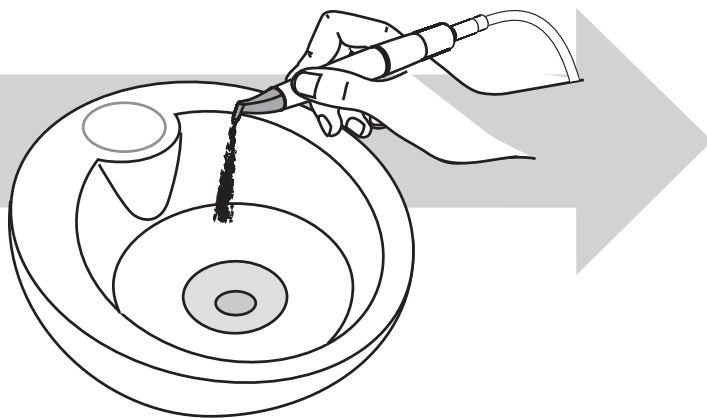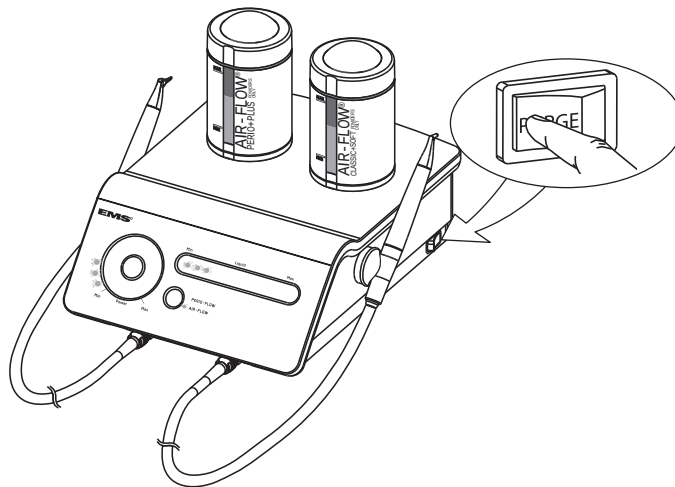
Purge switch and mode

Bouton et mode purge

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The purge switch allows to depressurize the powder chambers for safe removal. It also controls the "purge" function (see following pages).

Beeps will be emitted when the purge switch is pressed and the powder chamber lights will blink. After approximately 1 minute the unit will return to its previous configuration (or when pressing the switch a second time).

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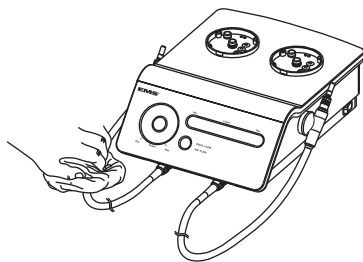
CLEANING, DISINFECTING AND STERILIZING

Please conform to the recommendations of the Reprocessing Instructions manual delivered with your product regarding procedure of cleaning, disinfecting, sterilizing and packing of the components.

Please follow present-day rules about sterilization applicable in your country.

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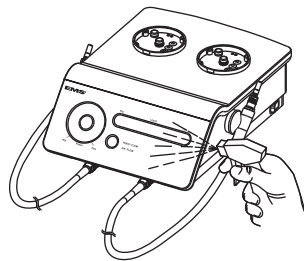
Unit

Clean the unit only with an alcohol-based, commercially-available, colorless disinfectant (ethanol, isopropanol). The use of scouring powder or an abrasive sponge will damage its surface.

Never remove the silicone caps from the powder chamber holders when cleaning the unit.

Do not use needles on the silicone caps of the powder chamber holders.

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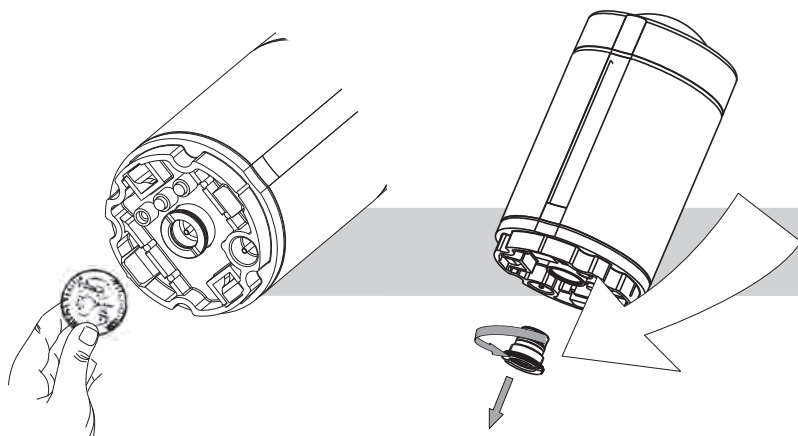
VERSION 2

 If the valve is damaged or worn out, it must be replaced immediately.

To replace the powder chamber valve:

- Do not turn the powder chamber upside down.
- Unscrew the valve assembly under the powder chamber with a coin, unclip the cap and extract the valve.
- Place the new valve in the correct position on the support and screw the assembly under the powder chamber.

 If an O-ring is damaged or worn out, it must be replaced immediately.

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

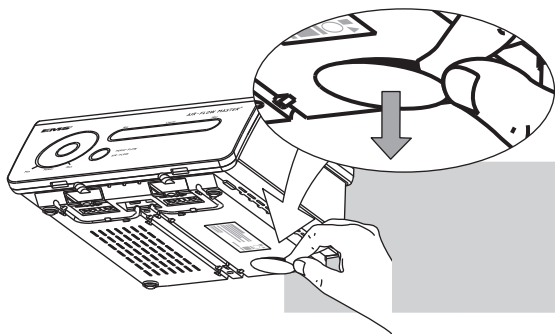
Filtre à air

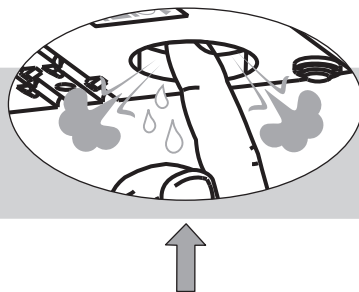Luftfilter

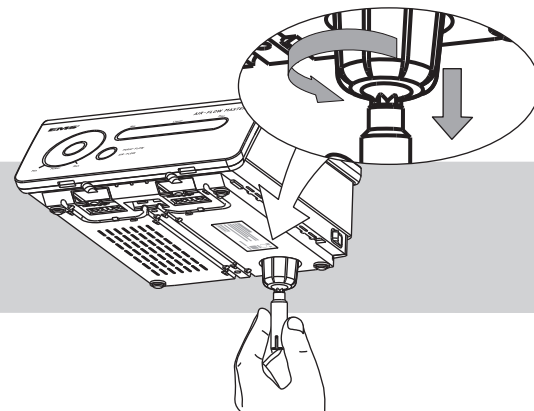Regularly control and purge the air filter.

The air filters are not reusable. Always install a new EMS filter.

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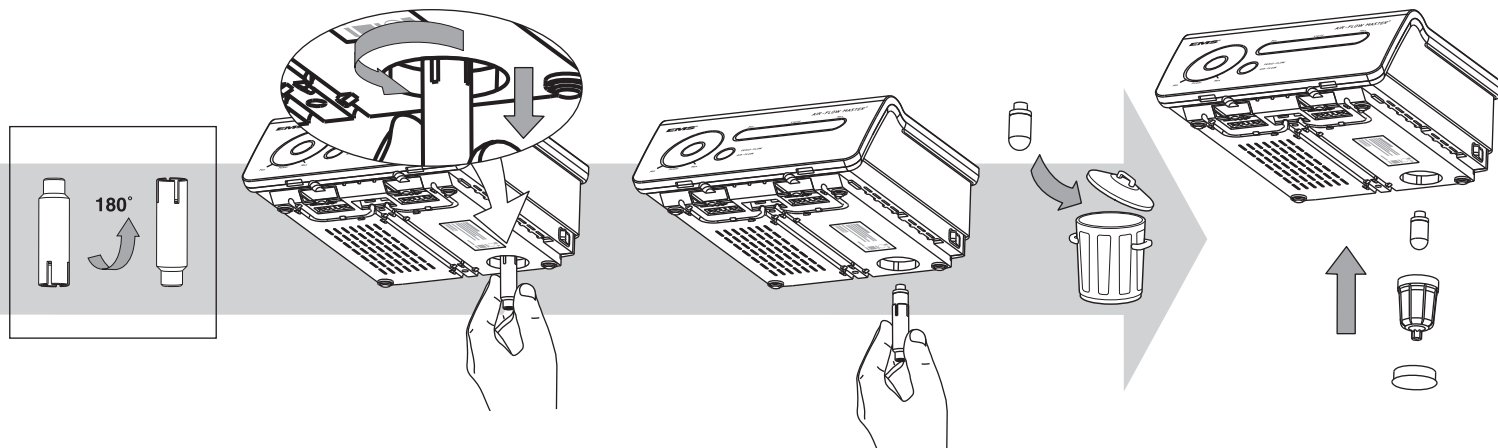
SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Use for the intended purpose only: before using the product, make sure that you have studied the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Before each use, clean, disinfect and sterilize the different pieces and accessories of this product. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Use only the powder recommended by EMS for the subgingival application when the PERIO-FLOW handpiece is connected to the unit. Any other powder could cause premature wear of the nozzle and would consequently increase the risk of emphysema and damage to the unenameled dentine.

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 This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated physiological effects.

 The powders provided by EMS are specially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use an abrasive powder i.e. based on silica or alumina. This would damage the unit.

 Close the powder chamber after use, risk of device obstruction

 Always examine the product for damage before starting treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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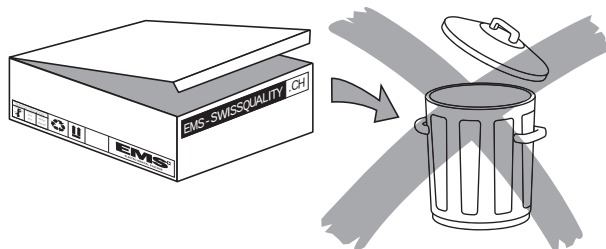
STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing",
- Remove the screw of the water filter to empty the unit,
- Pack the product and all accessories in the original packaging.

 The powder should not be stored near acids or heat sources.

Please refer to the storage and transport conditions in the chapter "Technical data".

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PRODUCT DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by EMS.

WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or to an authorized EMS Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

 Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW MASTER®
Classification EN 60601-1	- Class I - Applied part, Type BF - IP 20 unit - IP X1, foot pedal
Classification 93/42/EEC	Class IIa
Operating mode	Continuous operation
Supply voltage	100-240 VAC 50-60Hz
Power consumption	max. 100 VA
Fuses	T 1.6 A 250 VAC (Ø5 X 20mm, time lag, low breaking capacity)
Water supply	1 to 5 Bar (100 to 500 kPa)
Compressed air supply	5.5 to 7.5 Bar (550 to 750 kPa)
Weight (approx.)	5.0 kg
Dimensions in mm (HxWxD)	225 x 225 x 295
Operating conditions	+10°C to +40°C 30% to 75% relative humidity Altitude max. 3000 m
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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ELECTROMAGNETIC COMPATIBILITY

 This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the note "Electromagnetic compatibility".

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" must be taken into account.

 This product should not be used adjacent to or stacked with another unit. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

 The use of accessories and cables other than those specified or sold by EMS as replacement parts may result in increased emissions or decreased immunity of this product.

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TROUBLESHOOTING

Symptom	Solution
The unit does not start or stop	• Check the electrical connections and power supply at the mains • Check the fuses on back panel • Check main switch function: at least one LED must be visible on adjustments and working mode areas • The unit is in "standby" mode, tap on any sensitive zone to exit standby • Send the unit to an authorized EMS Repair Center
The foot pedal does not activate the unit	• Check that the foot pedal is properly connected • Replace the foot pedal • Send the unit to an authorized EMS Repair Center
The unit is still activated when the foot pedal is released	• The foot pedal is active, check for jam at foot pedal • The switch in the pedal is blocked. Turn the unit off at the power switch • Replace the foot pedal • Send the unit to an authorized EMS Repair Center
User interface displays an error: there is one acoustic signal and all LEDs are blinking	• Powder chamber pressure is too low or too high • The air or liquid regulator is switched off • Foot pedal and purge commands are not recognized by system. Check connections
The handpiece does not remain on the magnetic support	• Install the magnetic support on a metallic part of the unit • The support is demagnetized • Replace magnetic support
The handpiece does not hold on the handpiece cord	• Check for presence of the O-ring seal and replace if necessary • Replace the handpiece • Replace the handpiece cord • Send the handpiece and the handpiece cord to an authorized EMS Repair Center
No water jet coming from the handpiece	• Check the water supply and water filter • Check the water adjustment of your unit • Check the connections between the unit, the cord and the handpiece • Check for presence of water at the cord outlet port • Send the handpiece to an authorized EMS Repair Center
No powder/air jet coming from the handpiece	• Check the quantity of powder in your unit • Check the compressed air supply and air filter • Check for presence of powder/air at cord outlet port • Eliminate residual powder present in the channels using the cleaning needles provided • AIR-FLOW PERIO+ PLUS powder chamber only: remove the diffuser and empty the powder. Blow the diffuser with compressed air before installing it again and refilling. • Send the system to an authorized EMS Repair Center

Cont. next page

Water leakage between the handpiece cord and the handpiece	• Check the condition of the handpiece connecting O-rings • Send the system and the handpiece cord to an authorized EMS Repair Center
Water leakage between the handpiece and the nozzle	• Verify the position of the nozzle and its position on the handpiece • Verify that no residual remains between the nozzle and the handpiece • Verify that nothing affects the alignment or prevents fixing the nozzle on its holder. • Replace the nozzle • Replace the handpiece • Send the system and the handpiece cord to an certified EMS Repair Center
Air and powder leakage on the main unit	• Verify the positioning of the bottles • Check the O-rings under the silicone caps of the powder chamber holders • Verify the cap of the powder chamber and the O-ring positioning • Verify the check valve and the O-ring under the powder chamber • Remove all powder dust between the bottle holder and the powder chamber • Send the unit to an authorized EMS Repair Center
Air and powder is leaking at powder chamber	• Check the seal condition and the cleanness of the thread of the powder chamber cap • Replace the seals if necessary • Clean the sealing surfaces carefully • Replace the powder chambers • Send the unit to an authorized EMS Repair Center
Handpiece efficiency becomes weak or decreases during use	• Verify the quantity of powder in your unit • Eliminate residual powder present in the channels using a cleaning needle • Send the system and the handpiece cord to an certified EMS Repair Center
Leakage on the unit water supply	• Check the water supply connection and inlet pressure • Send the unit to an authorized EMS Repair Center

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EMS Italia S.r.l.
EMS Corporation